



Instructions for users of Comfort Raising aid Quick-Up



The Quick-Up aid fulfils, together with Comfort Bed System, the applicable parts of the requirements in the bed standard EN 1970:1999 and it is intended to be used as a handicap aid, for home care or similar. The Quick-Up aid is intended for individuals older than 12 years of age.






1. Instructions for use



The Quick-Up aid must NOT be used outside the bed's long side!

1.1 Placing the Quick-Up aid in user position

- When the aid is to be used, it is to be placed in the attachment so that it points slightly diagonally towards the bed's middle line. The small stop on the aid's tube has to be located in the attachment's deeper notch.

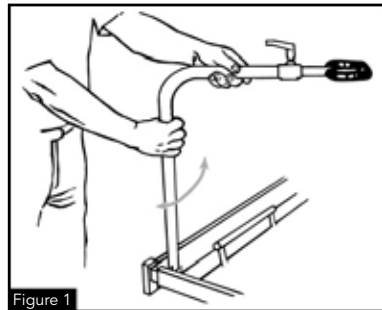

Check that the Quick-Up aid is in the right position by trying to turn it to the sides.


1.2 Length adjustment

- The aid can be adjusted to the correct length by loosening the lever as shown in **figure 2** and pulling the handle out to the desired length. NOTE! An inbuilt stop means it is not possible to pull the handle out completely.



1.3 When the aid is not in use

- When the aid is not in use, it can be placed parallel with the bed's foot end. Lightly lift the Quick-Up aid and turn it to the side as shown in **figure 1**. If the aid is to be returned to the user position – turn it in, over the bed, until it comes to a stop. Check once again that the Quick-Up aid is in the right position by trying to turn it to the sides.

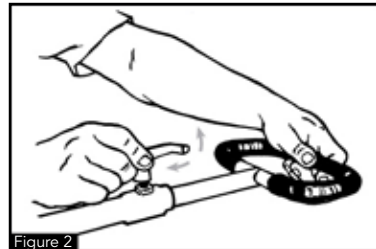

When the Quick-Up aid is used for moving, at least two of the wheels on the bed lift must always be locked.

2. Care instructions

The Quick-Up aid does not normally require regular maintenance. It can be cleaned with a damp cloth and readily available cleansers and disinfectants.

3. Field of application

The Comfort raising aid, type Quick-Up, may only be used together with the Comfort Bed System as an aid when moving in bed or when moving to/from bed.



Our experience - your guarantee

Since its start in 1987, Comfort System has kept more or less the same ownership and has operated under the same name – a fact that is almost unique today in the branch. Our long experience and knowledge of our product assortment, means that as a customer you can feel secure in your choice of supplier.



ComfortSystem

- our experience is your guarantee